

Battlefield Technology Meets Oral Wound Care

Introducing the HemCon[®] Dental Dressing for Grafts and Other Oral Surgeries



The 1" x 3" format is ideal for oral wound protection following:

- Free Gingival Graft Donor Sites
- Biopsies
- Extractions
- Oral Wounds
- Self-adhesive
- Dissolves in 48 Hours
- Seals the Wound Bed to Provide Pain Relief

From the makers of the revolutionary hemostatic HemCon[®] Bandages comes an oral wound care dressing that suits your surgical and extraction needs.

This pioneering oral wound dressing offers periodontists and oral surgeons an innovative dressing for soft tissue wounds, including palatal and gingival grafts and other oral surgeries. The flexible 1" x 3" dressing can be trimmed to custom fit most oral surgery sites. Fast, effective, and easy to work with, this medical wonder offers self-adhesion – **no sutures needed**. The HemCon Dental Dressing seals the wound surface with a protective layer, while reducing pain and chair time.



A Revolution In Oral Wound Dressings From HemCon Medical Technologies, Inc.



- 1" x 3" size – custom cut to fit
- No sutures needed
- Positive charged design – stabilizes the wound by attracting negatively charged red blood cells
- Self-adhesive
- Provides a physical barrier – seals the wound bed
- Offers pain relief
- Proven safe and effective for patients on anti-coagulants*
- Sterile and individually packaged in box of 12
- Ordering information at www.hemcon.com

Intended Use


The HemCon Dental Dressing is an oral wound dressing intended as a physical barrier for temporary protection of oral mucosal tissue and to provide pain relief.



Before Application

Steps for Use:

1. Custom cut the dressing to suit the patient's needs.
2. Using cotton pliers, place dressing onto surgical wound.
3. Dressing becomes sticky when in contact with liquid or blood.
4. Place sterile gauze over the dressing and apply pressure to seal to wound.
5. Check correct positioning of the dressing before discharge.
6. Dressing usually dissolves within 48 hours, may take up to 7 days. May be irrigated to remove any residual material.

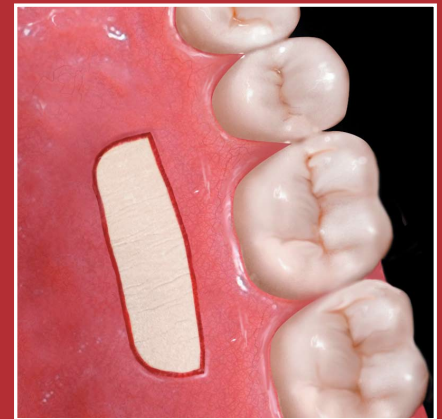

After HemCon Dental Dressing

HemCon Medical Technologies, Inc. was founded on one simple premise. SAVE LIVES.

The revolutionary hemostatic HemCon Bandages provide rapid control of moderate to severe external (arterial) hemorrhaging and provide an antibacterial barrier to promote healing with no adverse effects reported to date.

Since 2003, the real-time benefits are clear with HemCon Bandages saving hundreds of lives in Iraq and Afghanistan. As of September 2005, the U.S. Army Surgeon General mandated that all deployed U.S. Army soldiers carry at least one HemCon Bandage in their individual first aid kits, with medics carrying at least five HemCon Bandages. According to the U.S. Army, there is a reported 97% effectiveness on the battlefield.

Now the same battlefield technology bandage material used by the U.S. Army to control severe battlefield wound hemorrhaging comes to a dental dressing to serve your patient's needs.



HemCon
MEDICAL TECHNOLOGIES INC

10575 SW Cascade Avenue, Suite 130, Portland, OR 97223-4363
Phone: +1.503.245.0459 • Fax: +1.971.223.1000
www.hemcon.com • info@hemcon.com

*During clinical trials, patients on anti-coagulants (Aspirin and Coumadin) were treated with the HemCon Bandages. All dressings were effective.